




TENA *Slip Stretch* XXL


Bariatric Product Fitting Guide




Fitting TENA Slip Stretch XXL – Lying

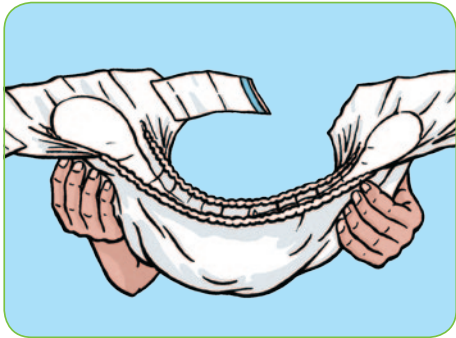


Fully open the pad and fold it lengthways into a bowl shape.

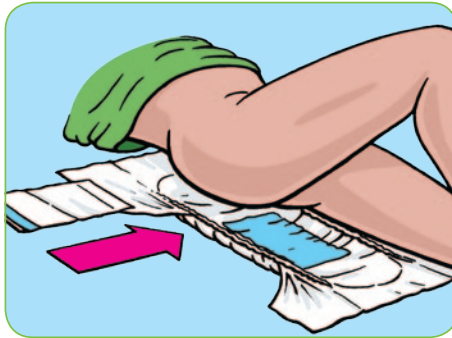


With the individual lying on their side and the pad at the back, position the pad with the fixation tabs in line with the hips.

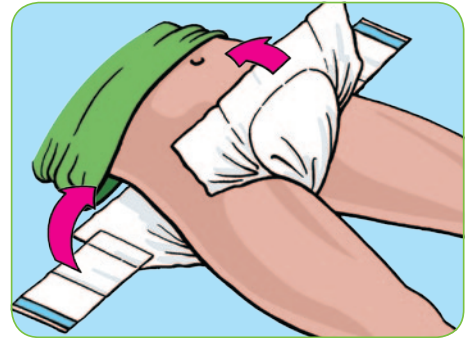

With the individual lying on their back, gently pull the pad between the legs.

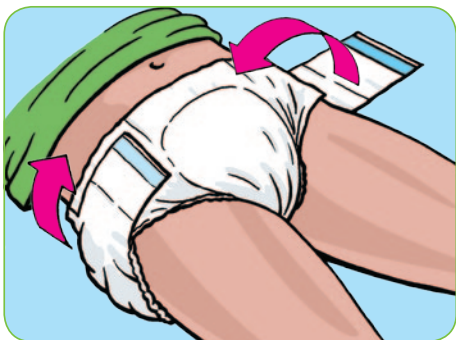


Fit the Wide Hook fixation tabs straight across the hips and fasten securely.

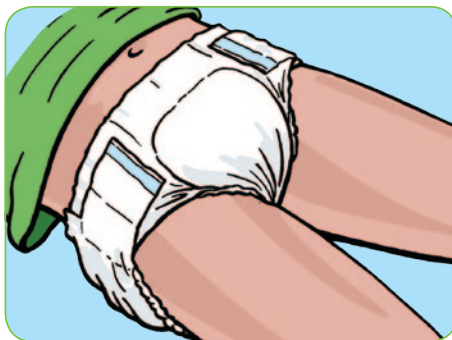


Check for comfort and fit. The pad allows for refastening and adjustment of the hook and eye fixation.


Fitting TENA Slip Stretch XXL – Standing

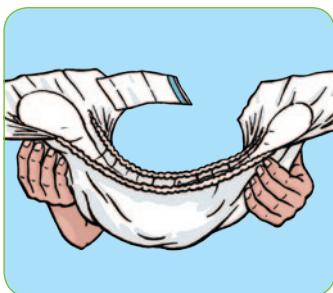

Fully open the pad and fold it lengthways into a bowl shape.

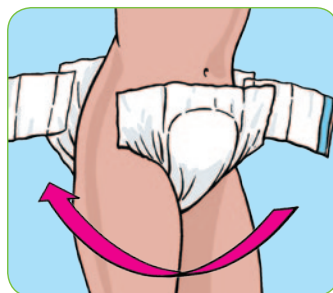

The end of the pad with fixation tabs is the back of the pad. With the individual standing, place the pad between the legs from front to back.

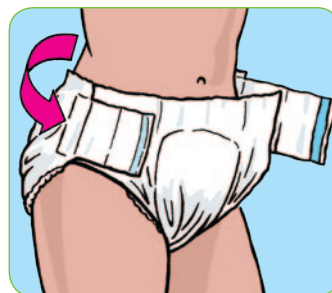

With the individual standing, fit the Wide Hook fixation tabs straight across the hips and fasten securely on both sides.



Check for comfort and fit. The pad allows for refastening and adjustment of the hook and eye fixation.



SCA HYGIENE PRODUCTS UK LIMITED,
Personal Care, Southfields Road
Dunstable, Bedfordshire LU6 3EJ
Tel: 01582 677400
www.TENA.co.uk

c/o Life Because our products make life easier for you and for millions of people around the world. Because our resources and the way we work are natural parts of the global lifecycle. And because we care.