

Abri-Flex (Pull-ups Pants)

The Abri-Flex pull-ups can be worn like normal underwear to give a more comfortable and secure fit. The soft and breathable backsheet offers greater comfort for the wearer and helps promote skin integrity. Leakage barriers ensure maximum protection and locks in any fluid in the pad. Wetness indicators are featured on these products which can help you to identify when the product needs changing.


Abri-Flex Fitting Guide – Standing position



Fold the pad lengthways with the backsheet turned outwards ensuring the inside surface of the brief isn't touched. Put on the brief as you would underpants.



After use and when changing, tear open the sides of the brief.

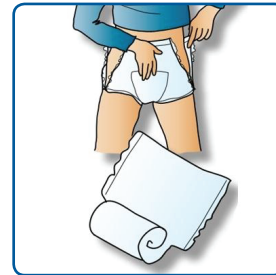

When both sides of the brief are torn open, remove the brief and discard it.

Abri-Flex dos and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Pull pad all way up
- ✓ Ensure a close fit is achieved
- ✓ Tear side-seam to remove pad
- ✓ Use tape to wrap pad for disposal
- ✓ Pads can be removed from their packets up to an hour prior to use, to deliver optimum performance
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, use sparingly to ensure the product works effectively