


# Attends For Men

## (Light Shaped Pads)

Attends For Men incontinence pads are especially shaped to fit the male anatomy. They are made of soft cotton like material and have a fixation tape which allows the product to be worn with your own close fitting underwear. The super-absorbent core delivers fast absorption to reduce the risk of leakage and provide skin dryness. The pads are white and very thin, with a soft feeling backsheet for ultimate discretion. Odour protection minimizes the risk of unpleasant odours.


### Attends For Men Fitting Guide

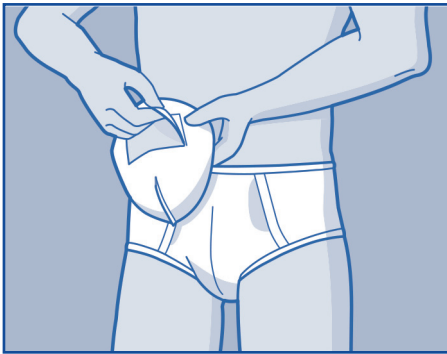

Ensure that the pad is folded out with the wider section of the pad facing upwards (as shown by the arrows on the release paper). Remove the release paper.



Secure the pad into your underwear making sure that it keeps its 'cup' shape.

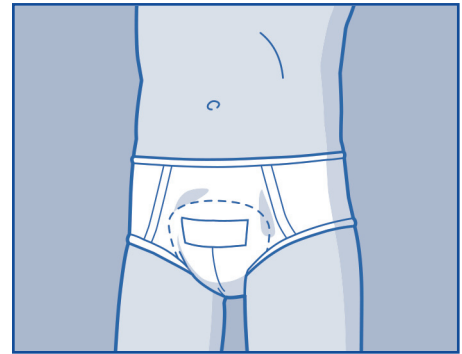

Fasten the pad with the fixation tape to the inside of your own close fitting underwear ensuring that the tape is not next to the skin.

### Attends For Men dos and don'ts

- ✓ Assess the user for correct absorbency
- ✓ Always fold pad lengthways to create bowl shape
- ✓ Ensure pad is fitting close to the body
- ✓ Pads can be removed from their packets up to an hour prior to use, to deliver optimum performance
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc as it reduces absorbency of the product
- ✗ When creams are prescribed, use sparingly to ensure the product works effectively
- ✗ The products are designed to be body worn, do not use as bed or chair protection