


Attends Pull-Ons


(Pull up Pants)

Attends Pull-Ons incontinence pants are an absorbent incontinent pant just like normal underwear! These pants can be pulled up and down easily when required but have the additional protection of built in incontinence pad maximising dignity and minimising hassle. The incontinence pants sit close to the body providing a secure fit and protection from leakage and when necessary they can be easily removed and should be hygienically disposed of. If needed the pad can be torn at the side for quick disposal. The flexible fit and breathable back sheet gives Attends Pull-Ons the feeling of normal underwear, increasing the feeling of freedom. Added odour protection quickly locks away any unpleasant smells.

Attends Pull-Ons Fitting Guide

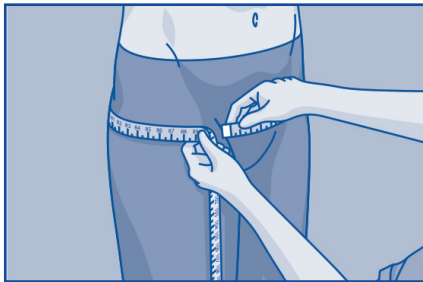


Measure the hip and waist and using the larger of these two sizes ensure that the correct product size is selected.

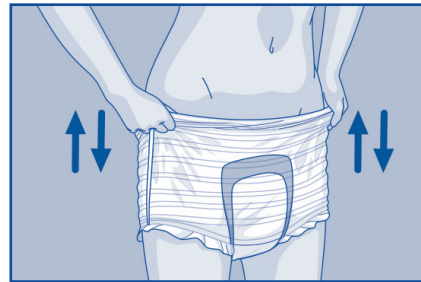

Attends Pull-Ons can be used like ordinary underwear and the pad can be pulled up and down as many times as you like.



The elasticity of the pants ensure a snug, comfortable fit whilst the breathable material is skin friendly.

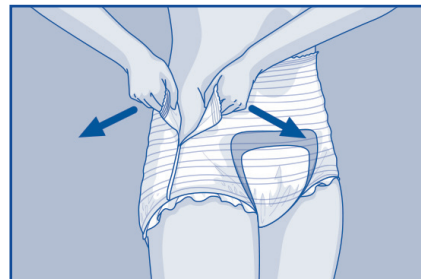

Changing and removing the pad is very easy: either pull the pant down or you can tear open both sides of the pant.

Attends Pull-Ons dos and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Pull pad all way up
- ✓ Ensure a close fit is achieved
- ✓ Tear side-seam to remove pad
- ✓ Use tape to wrap pad for disposal
- ✓ Pads can be removed from their packets up to an hour prior to use, to deliver optimum performance
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, use sparingly to ensure the product works effectively