


Lille SUPREM Fit


(All-in-one Pads)

The SUPREM Fit All-in-One briefs offer an effective solution for managing moderate to severe incontinence. With a high level of absorption, the SUPREM Fit all-in-ones provide the users anti-leakage security and optimal comfort. They are particularly recommended for people with limited mobility or confined to bed.

Lille SUPREM Fit Fitting Guide – Lying

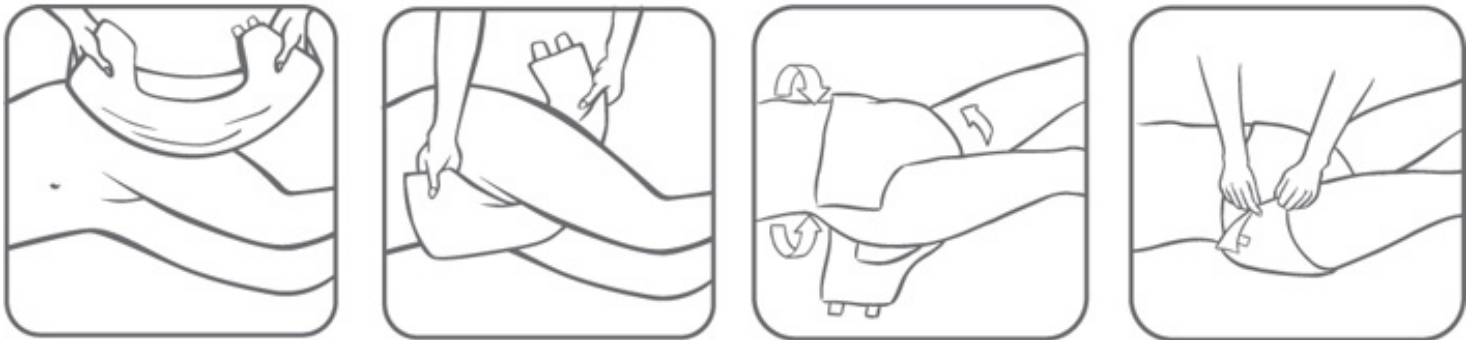

Lille SUPREM Fit Fitting Guide – Standing

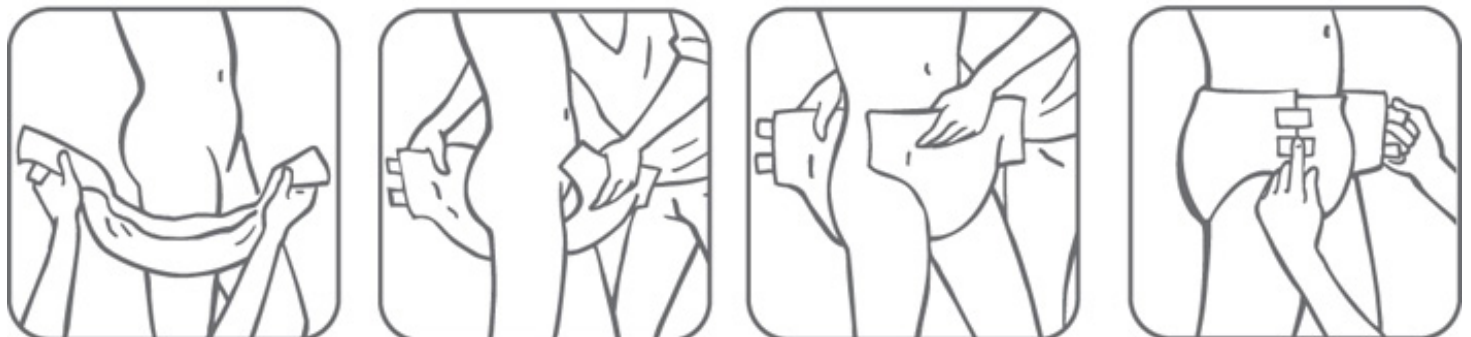

Lille SUPREM Fit dos and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Fold pad lengthways to create bowl shape
- ✓ Fit from front to back
- ✓ Seal bottom tabs pointing upwards and top tabs pointing downwards
- ✓ Ensure standing gathers around leg openings are pulled up before fitting
- ✓ Ensure a close fit is achieved
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, used sparingly to ensure the product works effectively
- ✗ Never 'twist' the product
- ✗ Never shake the product in order to open