

Lille SUPREM Form

(Shaped Pads)

The SUPREM Form shaped pads are a range of high quality products. They are an effective solution for managing moderate to severe incontinence. All the products within the range provide high levels of absorption. The user feels dry and comfortable. The SUPREM Form shaped pads can be used with fixation pants or close fitting underwear.

Lille SUPREM Form Fitting Guide – Standing



Lille SUPREM Form Fitting Guide – Lying



Lille SUPREM Form dos and don'ts

- ✓ Assess the client for correct absorbency
- ✓ Measure client around the waist for correct fixation pant size
- ✓ Always fold pad lengthways to create bowl shape
- ✓ Insert pad from front to back
- ✓ Ensure pad is fitting close to the body with wider part to the back
- ✓ Fixation pants should always be used with Lille SUPREM Form range
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc as it reduces absorbency of the product
- ✗ When creams are prescribed, use sparingly to ensure the product works effectively
- ✗ The products are designed to be body worn, do not use as bed or chair protection