

TENA Pants (Pull up Pants)

TENA Pants Discreet

A soft, comfortable, disposable absorbent product that can be handled just like normal underwear, encouraging greater independence. TENA Pants Discreet offers superb security and discretion. They are thin, soft and stretch to the shape of the body.


TENA Pants Normal

TENA Pants provides outstanding security through FeelDry™ technology, which rapidly absorbs even large quantities of urine into the product core and quickly away from the skin, keeping the surface exceptionally dry. The product range also incorporates Body-Close Fit™, resulting in increased discretion and an improved sense of dignity. TENA Pants offer outstanding dryness, comfort and healthier skin for the user.

TENA Pants Plus, Super & Maxi

TENA Pants Plus, Super and Maxi feature ConfioFit technology, which provides great comfort and discretion due to its thin, anatomically shaped protection. The higher absorbencies offer added protection where it is needed. High breathability also makes TENA Pants Plus, Super and Maxi a great choice to promote skin health.

TENA Pants Fitting Guide

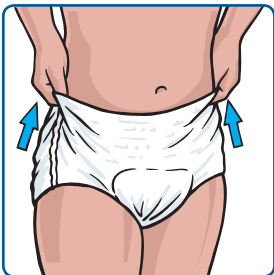

1. Pull up just like normal underwear.
2. Tear at the sides for easy removal.

TENA Pants dos and don'ts

- ✓ Measure around the hips for correct size
- ✓ Assess for correct absorbency
- ✓ Pull pad all way up
- ✓ Ensure a close fit is achieved
- ✓ Tear side-seam to remove pad
- ✓ Use tape to wrap pad for disposal
- ✓ Pads can be removed from their packets up to an hour prior to use, to deliver optimum performance
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc, as it reduces the absorbency of the product
- ✗ When creams are prescribed, use sparingly to ensure the product works effectively