



iD for Men (Light Shaped Pads)

iD for Men are individually wrapped pads ideal for managing light to moderate urinary incontinence in men. The male-specific design enhances comfort and security the shape and adhesive strip ensure a secure fit in conventional underwear.


iD for Men Fitting Guide

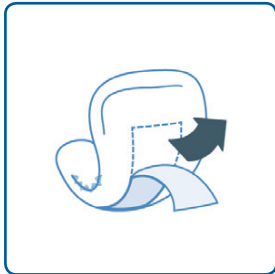

Unfold the iD for Men pad and remove the paper from the adhesive strip.

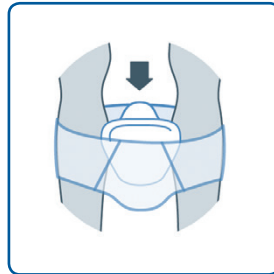

Ensuring the wider part of the product is at the front, place the pad inside the underwear making sure the adhesive strip is securely fastened to the underwear.

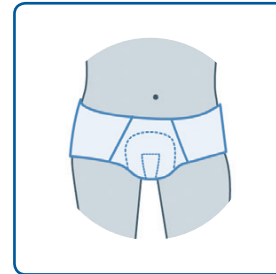

Finally, pull up the underwear and adjust for comfort and security. It is not recommended that iD for Men pads are worn in loose fitting underwear, for example boxer shorts. Good, close fitting, supporting underwear is required to ensure full product efficiency.

iD Expert Light dos and don'ts

- ✓ Assess the user for correct absorbency
- ✓ Always fold pad lengthways to create bowl shape
- ✓ Ensure pad is fitting close to the body
- ✓ Pads can be removed from their packets up to an hour prior to use, to deliver optimum performance
- ✗ Never place one pad inside another to increase absorbency
- ✗ Never use talc as it reduces absorbency of the product
- ✗ When creams are prescribed, use sparingly to ensure the product works effectively
- ✗ The products are designed to be body worn, do not use as bed or chair protection